ELBER

M E D I C A L

MANUAL DE INSTRUÇÕES E GARANTIA
INSTRUCTION AND WARRANTY MANUAL
MANUAL DE INSTRUCCIONES Y GARANTÍA

CAMARA CONSERVADORA DE SEMÊN SUÍNO ELBER

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25	BATTERY	36
26	BETTER USE OF CONSERVATIVE AND RECOMMENDATIONS	36
27	ICE AND CLEANING	37
28	MAINTENANCE AND PREVENTIVE CHECKING	37
29	EMERGENCY SYSTEM <i>(see optional availability)</i>	38
30	ELECTRONIC SELF DIAGNOSIS	38
31	PROBLEMAS E CAUSAS PROVÁVEIS	39
32	DISPOSAL OF PRODUCT	39
33	WARRANTY	40
34	18 HANDLING PROCEDURE TELE ALARM 6 NUMBERS <i>(see option availability)</i>	40
34.1	Register phone numbers	40
34.2	Deleting Memory Numbers	41
34.3	Test the tele alarm memories	41
35	HANDLING PROCEDURE TELE ALARM 9 NUMBERS <i>(see option availability)</i>	42
35.1	Registering phone numbers	42
35.2	Programming a Paused Phone Number 2 sec. (for PBX calls)	42
35.3	Programming a Telephone Number for a Call Through a Economy Line	43
35.4	Programming a Long Distance Calling Phone Number	43
35.5	Deleting a Phone Number	43
35.6	Testing the Phone Numbers	43

35.7	Changing the Installer Password	43
35.8	Dialing Process	43
35.9	Dialing Interrupt Modes	44
35.10	TOTAL RESET (All dialer parameters return to Factory Default)	44
35.11	Audible Signaling (Buzzer)	44
36	GSM HANDLING PROCEDURE (see option availability).	45
36.1	Installing the SIM CARD	45
36.2	Signals	46
36.3	Programming the GSM Dialer	46
37	WIFI MODULE HANDLING PROCEDURE (see option availability)	47
37.1	Configuring the WiFi module	47
37.2	History	48
37.3	Settings	48
37.4	Register mobile phones and emails	48
38	CARACTERÍSTICAS CONSERVADORAS	49
38.1	Embalaje	50
38.2	Transporte	50
38.3	Almacenamiento	50
38.4	Manipulação	50
39	ESPECIFICACIONES TÉCNICAS	51

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DEAR USER!

You have just purchased a conservative for swine semen storage. This product was developed with ELBER technology and functionality. We are grateful for your choice.

It is essential to carefully read the instructions contained in this manual.

By following these guidelines, you can better understand how the product works, enjoying its benefits and extending its durability. The instruction manual should always be accessible near the product so that any questions can be remedied immediately.

19 CONSERVATIVE CHARACTERISTICS

The conservatives of the CSA line were developed to conserve and stir porcine semen. Store samples at a stable, homogeneous and uninterrupted temperature. The mechanism that moves the grids is automatically activated, shaking the samples for 3 minutes every 6 hours (these times may vary up to 15%).

Conservatives operate on 12 VDC battery (*). The battery charger runs on commercial 110-127 VAC or 220-240 VAC power, so the conservative must remain constantly plugged in to keep the battery charged.

The conservatives in the CSA range are designed to ensure that the internal chamber temperature is always stable and homogeneous. Equipped with a security system, the conservative Elber remain operating even in the absence of commercial electricity (*). Conservatives have a microprocessor electronic controller that controls the chamber's internal temperature (cooling and warming as needed) and informs the user of all events through audible and visual alarms and electronic logs.

Attention: The battery does not come with the product. Always use a new stationary battery.

(* Exception: If the product does not have a battery charger, it indicates that it has no emergency system, so it does not need a battery to function.

Homogeneous Temperature: Through a system that adds integrated ventilation, cooling and heating, which operate in an integrated manner, it allows a temperature of the same intensity anywhere within the chamber. The system warms the chamber when there are cold days and cools when there are hot days.

Grill Shaker: It is automatically triggered every six hours and keeps the grids moving for three minutes to move semen samples (these times can vary up to 15%).

19.1 Packing

The product packaging you received was designed to provide safe shipping for the conservative. When unpacking, check the integrity of the product. ELBER conservatives are packaged with bubble wrap and cardboard. Some products have MDF or wooden pallet packaging. Each product is individually packaged.

After unpacking the product, any and all packaging material should be sent for recycling or reuse later.

19.2 Transportation

ELBER conservatives must be transported in an upright position, keeping the packaging conditions in accordance with the label.

19.3 Storage

ELBER conservatives should be stored in a clean, dry environment, protected from sunlight and excessive humidity.

Conservatives equipped with an internal battery should not be stored for longer than two months without use. If left unused, the battery will discharge.


19.4 Handling

ELBER conservatives should only be handled by trained professionals who fully comply with the specifications in the product instruction manual.

20 TECHNICAL SPECIFICATIONS

Model	CSA 175	CSA 370
External Dimensions (mm)		
Height	1408	1578
Width	480	638
Depth	609	775
Internal Volume (L)	175	370
Weight (kg)	73	138
Consumption (A)		
110 V	3,0	4,0
220 V	1,5	2,0
Internal temperature	+15°C a +18°C	

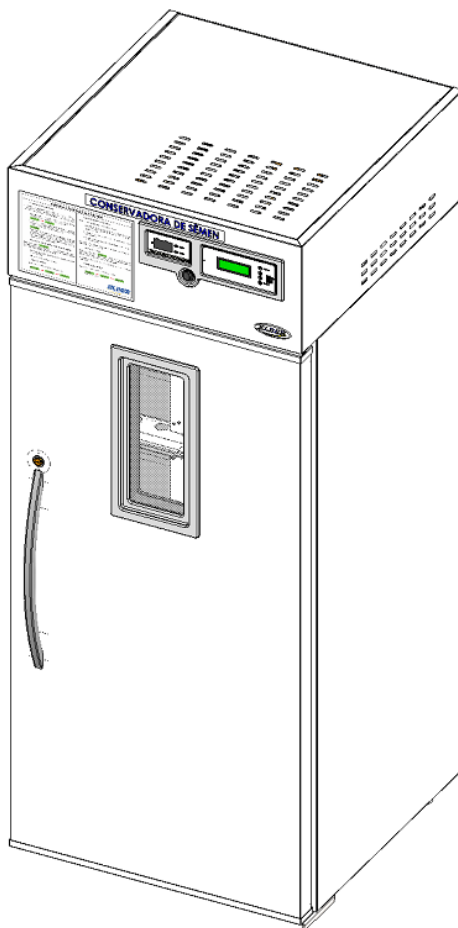The manufacturer reserves the right to modify the general, technical and aesthetic characteristics of its products without prior notice.


21 CONSERVATIVE INSTALLATION AND FIXING

The conservative should be positioned on level and level ground. Some models have casters and other leveling feet. In conservatives with casters, to move the product, the casters must be unlocked and locked when the conservator is in position. Moving the conservative with the casters locked may damage the casters and the floor. For conservatives with leveling feet, adjust the level after positioning the product.

For conservatives equipped with a water drain, connect the hose to a drain or drain hose.

The conservative Elber should be installed preferably protected from the sun and in a place where there is air circulation. Very small rooms should have air exchange openings.

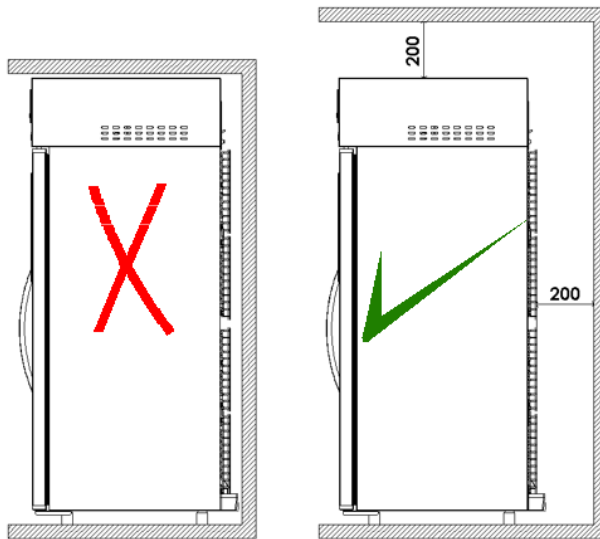We do not recommend installing the conservator near heat sources (heaters and greenhouses). In the conservator's ventilation grilles or in the static condenser area (rear), there should be at least 20 cm clearance from the wall or any obstruction to allow ventilation of the compressor unit and the electronics.

22 ELECTRICAL INSTALLATION

The conservator is ready to operate on 110-127 VAC or 220-240 VAC power.

Before turning on the conservator, check that the voltage indicated on the product matches the mains voltage.

The power cord must be plugged into a permanent wall outlet or workbench in accordance with current regulations. Do not use extensions or adapters.

Observe the following steps to properly connect the conservative.

1. Make sure the conservative main switch is off;
2. Connect the battery cables to a new stationary battery. Securely tighten the necectors to the battery poles (not loose). Note the polarity of the wires, which must be respected. In conservative the blue cable is (+) and the black cable is (-) (the cables are also marked with washers). This step should be disregarded if the conservative does not have a battery charger;
3. Plug the conservative plug into the outlet;
4. Turn on the conservative main switch;

When you turn on the conservative, the Buzzer will sound until the temperature inside the camera is within the ranges set on the digital controller.

You can cancel the audible alarm for a preset time by pressing any key on the controller. When the alarm is sounding and deactivated, it will sound again after the set time if the temperature remains outside the programmed range. The alarm will sound until the conservative reaches normal operating range (on average 2 hours without load).

To switch off the conservative, when equipped with emergency system (battery), it is necessary to turn off the main switch. It is not enough to remove the plug from the socket as the conservative will continue to operate on battery power (*see option availability*).

We recommend using the SPW 275-12 - WEG Surge Protection Device (DPS). This device protects the conservative against surges and over voltages, preventing it from burning.

Attention!

This conservative should be grounded to prevent damage to the product and the user. Grounding is required at any voltage (110-124 VAC or 220-240 VAC). Lack of grounding compromises conservative warranty coverage. For grounding and electrical installation, follow ABNT-NBR 5410 standards or call a responsible professional.

23 OPERATION

This conservative is prepared for the preservation of porcine semen.

Guarantees a constant temperature at any time of the year and works even when there is a power outage (*see optional availability*). The temperature is automatically adjusted and controlled by the system and there is no need for intervention even on cold or hot days.

A uniform temperature is guaranteed by the digital controller that keeps what has been programmed. The controller is programmed to maintain the camera temperature between **+ 15.0 ° C** to **+ 18.0 ° C** on both cold and hot days. At any time of the year, winter or summer, the conservative will adjust automatically.

The temperature will always remain within a defined range, ensuring the best properties of the conserved material.

23.1 Grate stirring mechanism

The grid agitation mechanism is activated every six hours and causes the grid to remain in motion for three minutes to move semen samples.

Each time the conservative is energized, a cycle begins, so after six hours, the crates will move for three minutes.

The times quoted above can vary up to 15%.

23.2 Grid movement cycle counter

The cycle counter helps to highlight the operation of the product. Each time the crates move, perform 1 cycle. Therefore, at the end of 6 hours of rest, a 3 min stir cycle was started and the counter records the 1 cycle count.

To reset the counter, press and hold the “RESET” button for 10 seconds.

23.3 Test Button

To check the mechanism operation, press the “TEST” button on the cycle counter to activate the grids.

Pressing the button will move the grids and start a new cycle. It is advisable to test the mechanism operation each time the conservative is turned on.

Attention!

Take care of the grid stirring mechanism:

- **Do not open the door while the mechanism is engaged (when opening, movement is stopped by the electronic controller).**
- **Do not remove the conservator top cover while the mechanism is engaged.**
- **Only trained personnel should handle the conservative and perform maintenance.**
- **Do not perform maintenance with the conservator on (disconnect the power and the main switch, so that the conservative does not operate on the battery).**
- **Do not store miscellaneous items inside the conservative, use only the internal spaces of the semen ampoule storage crates;**
- **Internal grid locks must be tightly closed before turning on the conservator. If a lock is left open, the system may lock and component breakage may occur;**
- **If the grid moving system locks up, does not force the grids to return to the home position, this will cause the mechanism to break. In case of grid locking, contact AT Elber;**

24 BEEP ALARM

For convenience, the conservative has an audible alarm that sounds when the temperature is outside of programmed limits or other anomalies. This may occur if the door is left open for a long time or due to malfunction. Thus, in a timely manner, some action can be taken to ensure that preserving products are not lost. The alarm will be activated only after the predetermined time with temperature outside the programmed.

Alarm delay function: When the conservative is turned on the alarm will sound. To suspend the alarm for the preset time, press any key on the digital controller. Repeat as many times as necessary until the conservative reaches the programmed range where it will no longer fire. To change the time between triggers, see regular function time to trigger the alarm.

25 BATTERY

If the product has an emergency system, it is important to follow the recommendations below:

- For conservatives with emergency system and are purchased without the battery, always use a new battery.
- For static conservatives use stationary battery;
- When using unsealed battery, check distilled water level quarterly.
- Check battery operation. If it is not working properly, it will cause the conservative to malfunction.

26 BETTER USE OF CONSERVATIVE AND RECOMMENDATIONS

- Avoid leaving the door open.
- When in use, always keep products inside the conservator, even if they are bottles with water. This load of products helps to keep the temperature inside the conservative stable.
- Before placing products in the conservative check that the operation is correct and the indoor air circulation fan is working.
- Avoid leaving the conservator door open for a long time or opening it constantly;
- It is recommended to keep the conservator running for at least 48 hours before placing products in the conservator to verify that the overall operation is correct, that the internal fan is circulating air and that the temperature is within the set parameters;
- When in use, always keep products inside the conservator, even if they are bottles with water. This load of products inside helps to keep the temperature inside the conservative stable;
- Internal ventilation fins must not be obstructed, constantly check if stored products are obstructing the fins and therefore internal ventilation of the product;
- Shelves or drawers must not be lined with plastics, towels or the like, which compromises the internal ventilation of the product;
- On very humid days there may be moisture condensation (sweat) on the door or on the outer edges of the conservator (near the gasket). In places with room temperature controlled by air conditioning, this feature is virtually extinct.

- For conservatives equipped with an emergency system, do not perform battery runtime tests without Elber, face-to-face or online support. Only technicians can ensure the recording of results, disable the messaging system and ensure that the battery is undamaged (see option availability).
- Only trained personnel should operate the product.

27 ICE AND CLEANING

To clean the conservative, first turn off the GENERAL SWITCH. It is not enough to unplug it, as it will continue to operate on battery power (see option availability).

Use damp cloths with water and mild detergent. Do not use water nozzles or hoses, excess water may compromise the operation of the product.

Do not use chemicals (alcohol, solvents, etc.) or abrasives and hard sponges. Unsuitable procedures may stain and damage refrigerator surfaces and linings.

During cleaning, remove all materials stored in the conservator and place them in other equipment or thermal box with cryogenic material.

Important! After a conservative period of inactivity (for example, cleaning), on power up, the conservative may reach temperatures below the programmed temperature in the electronic controller. Therefore, every time you turn off the conservative after reconnecting, wait at least 2 hours to load the conservative again. After 2 hours, clear controller data and start normal use.

28 MAINTENANCE AND PREVENTIVE CHECKING

- Monthly Check that the door gasket seal is correct (there should be no cracks between the gasket and the cabinet);
- Monthly Check the level of glycerol present in the bottle, located inside the conservative (variable position depending on the model, some models do not have the sensor immersed in glycerol). If the sensor is out of solution, complete the level;
- For conservatives equipped with emergency system, check battery charge every six months (*see option availability*);
- Annually the conservator condenser must be cleaned. The location of the condenser varies for each model, but is always near the compressor or at the rear of the product;
- Every 2 years change the drive belt of the grate movement system;

- Every 2 years, change the drive motor brushes of the grate movement system;
- Change conservative evaporator and condenser fans every 4 years;
- For conservatives equipped with emergency system, replace conservative battery every 4 years (*see option availability*).

29 EMERGENCY SYSTEM (*see optional availability*)

When equipped with an emergency system, in the absence of conventional 110-127V or 220-240V power, Elber conservatives continue to operate, keeping all their electro-electronic, mechanical and cooling functions in operation.

Emergency system activation is automatic. The emergency system autonomy varies for each model, can be increased or decreased as the customer needs.

Conservative Elber have automatic charger. The recharge time of batteries varies between 10 and 20 hours.

Important: Conservative autonomy depends on many factors, such as: battery capacity used, ambient temperature, conservative charge, number of times the door is opened, among others. Refer to a 110 Ah automotive battery, which should have a range of approximately 10 hours (considering ambient temperature of $22 \pm 2^{\circ}\text{C}$, few door openings, new conservative automotive battery with full charge already at standard temperature). The higher the battery capacity, the longer the conservative will last.

he emergency system is an optional, check the availability of the optional.

Some models may be sold with the emergency system, but without the battery.

30 ELECTRONIC SELF DIAGNOSIS

LED	Flashing	Diagnosis
Flashing	1 time	Battery protection cut (voltage out of specification)
Flashing	2 time	Too many consecutive attempts to shorten compressor or fan (current over 0.5A)
Flashing	3 time	Compressor did not break or excess pressure. Wait 4 minutes for it to automatically reconnect.
Flashing	4 time	Minimum compressor speed error
Flashing	5 time	Ambient temperature too high
Flashing	6 time	Thermostat shorted or improperly connected to electronic unit.

31 PROBLEMAS E CAUSAS PROVÁVEIS

FAILURE	SOLUTION
Conservative doesn't care	<ul style="list-style-type: none"> - Check if the supply voltage is the same as conservative; - Check if there is electricity in the power supply; - Check if the main switch is on; - Check if the power plug is well connected; - Check if the protection fuse is blown; - Check if the temperature set in the controller is correct; - Check if there is any warning on the digital controller.
Conservative does not cool;	<ul style="list-style-type: none"> - Check if the compressor is on (place your hand over the condenser and check for a slight heating); - Check if the internal ventilation is working; - Check the closing of the door (door gasket cannot have cracks in relation to the cabinet); - Check if there is any warning on the digital controller.
Conservative alarming for high temperature	<ul style="list-style-type: none"> - Check if a large load of product has been charged less than 2 hours (conservative may take until it reaches the ideal temperature when a large amount of product is placed at one time); - Check door closure (door gasket cannot have cracks in relation to the cabinet). - Check if there is any warning on the digital controller.
Conservative internal temperature varying.	<ul style="list-style-type: none"> - Check if the internal fans are working; - Check if the internal ventilation fins are free (must not be obstructed);
Stirring mechanism does not turn on	<ul style="list-style-type: none"> - Check if the conservative is properly connected; - Visually check (without forcing manually) that the internal mechanism is locked; - Visually check that the mechanism has activated the limit switches (if yes, activate AT Elber); - Check if the protective fuse in the drive motor is blown;

32 DISPOSAL OF PRODUCT

Discarded or abandoned refrigerators can be dangerous, especially to children, as they can become trapped inside them and risk shortness of breath. Before disposing of your old refrigerator, cut off the power cord and remove the door (s). Leave the shelves in place so that children cannot enter.

If the conservative battery is disposed of, forward the discarded battery to a closer collection point.

In case of product disposal, all materials should be sent for recycling.

33 WARRANTY

Conservative warranty is 1 year.

Attention: The warranty term of the battery is enclosed with it, and cannot be misplaced as it contains the serial number that coincides with the battery (see option availability).

34 18 HANDLING PROCEDURE TELE ALARM 6 NUMBERS *(see option availability)*

The tele alarm operates through a telephone line. It makes calls to the registered numbers when the temperature exits programmed range, or if the battery voltage is low. For the dialer to work, you need to connect a telephone cable.

34.1 Register phone numbers

The tele alarm has memory to register up to 6 phone numbers, with up to 16 digits each.

To register a phone number, press the "PROG" key to enter programming mode. Enter "1", which corresponds to the desired function, and then press the number of the memory the phone number should occupy, from 1 to 6. Enter the phone number and press "ENTER" to confirm. Press "ENTER" again to exit programming mode. Ex.:

When the line has a PABX extension, press "0" to pick up the external line, and the "PROG" key to enter a 2-second pause before the telephone number.
 Ex.:

If a number is stored in an already occupied memory, the new number will replace the previous one, which will be deleted automatically.

We recommend that you record numbers from at least two different carriers, if the service provider fails and you can not perform one operation, the dialer can complete the call to another.

In case of tele alarm triggering, it will call all the numbers, according to the registration sequence. The dialer will continue to make the call sequence while there are shots.

34.2 Deleting Memory Numbers

Press the "PROG" key to enter programming mode. Enter "2", which corresponds to the desired function, and then press the number of the memory that corresponds to the phone number you want to delete (from 1 to 6).

Press "ENTER" to confirm, and again to exit programming mode. Ex.:

34.3 Test the tele alarm memories

Press the "ENTER" key and then the memory number corresponding to the phone number you wish to test. The LED will blink twice, ascend, and initiate the test call. If the memory is empty, the LED will rise for 1 second and the test will be completed. Ex.:

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To cancel the test press the "PROG" key.

THE OTHER FUNCTIONS OF THE TELE ALARM ALREADY COME PROGRAMED FROM THE FACTORY, NOT NECESSARY TO CHANGE.

35 HANDLING PROCEDURE TELE ALARM 9 NUMBERS (see option availability)

The alarm phone works over a telephone line. Calls the registered numbers when the temperature is outside the programmed range, or if the battery voltage is low. For the dialer to work, you need to plug in a phone cord.

35.1 Registering phone numbers

Press # + Installer Password + Index (01 to 09) + Phone Number (up to 22 digits) + # + #.

35.2 Programming a Paused Phone Number 2 sec. (for PBX calls)

Press # + Installer Password + Index (01 to 09) + 0 + E + Phone Number + # + #.

If you need to use the characters * (asterisk) or # (pound), proceed in the same way by pressing respectively the P or F key in the position where they are to be entered.

35.3 Programming a Telephone Number for a Call Through a Economy Line

For calls of this type, where you need to enter * 15 seconds for a 6-second pause before dialing, press # + Installer Password + Index (01 to 09) + P + 15 + E + E + E + Phone Number + # + #.

Phone numbers longer than 22 digits are not allowed. When the 22nd digit is entered, the system automatically records its value and does not need to press # to store it.

35.4 Programming a Long Distance Calling Phone Number

Press # + Installer Password + Index (01 to 09) + 0 + Operator Number + Phone Number + # + #.

35.5 Deleting a Phone Number

Press # + Installer Password + Number Position you want to delete (Index 01 to 09) + * + # + #.

Another way to delete a preset number is to access the desired index and omit the phone number value by pressing #. Be careful when accidentally accessing phone numbers (01 to 09) as you will need to re-enter their value.

35.6 Testing the Phone Numbers

With the telephone line correctly connected and the dialer in user mode (out of programming), press F + desired Memory Location (1 to 9 - without digit 0).

The dialer will make a call to the stored number, playing a siren sound on the phone for 60 seconds. During this time, the LINE LED will remain lit. To stop an ongoing test, press any key.

35.7 Changing the Installer Password

The installer password is for accessing Programming Mode and its factory default is 4321. To change, press # + Installer Password (current) + 10 (index) + New Installer Password (4 digits) + #.

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35.9 Dialing Interrupt Modes

Concurrent Mode: The dialer will make calls only while the alarm or fence goes off.

Persistent Mode: The dialer will make calls even if the trigger no longer occurs, completing the 2 cycles.

Press # + Installer Password + 12 (index) + 1 (Concurrent Mode) or 2 (Persistent Mode) + #.

35.10 TOTAL RESET (All dialer parameters return to Factory Default)

Turn off the dialer power and hold down the 7 and 9 keys. Temporarily reset the dialer until you hear 3 shorts beeps. Release keys 7 and 9 and power up the dialer.

35.11 Audible Signaling (Buzzer)

1 Long beep = Error tone (incorrect command or procedure).

1 Short Beep = Indicates a key press.

3 Short Beeps = Confirmation Tone (command or procedure accepted).

THE OTHER FUNCTIONS OF THE TELE ALARM ALREADY COME PROGRAMED FROM THE FACTORY, NOT NECESSARY TO CHANGE.

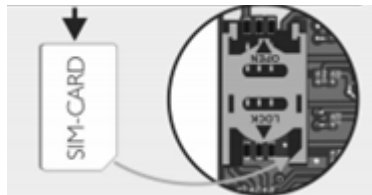
36 GSM HANDLING PROCEDURE (see option availability).

The conservative Elber can be equipped with GSM dialer, which sends SMS messages and makes calls in case of events. Below are the main instructions related to this item.

36.1 Installing the SIM CARD

Remove the front cover of the GSM dialer by pressing a small latch on the bottom of the carton. When removing the front, do not disconnect the cables. Choose a SIM CARD from an operator of your choice and enter as indicated below:

- Only the MICRO SIM CARD model can be used.
- Never remove or place the SIM CARD with the GSM dialer on.
- The SIM card does not come with the dialer. Choose an operator that you trust because the operation of the device is directly related to the cellular telephone network in your region.

36.2 Signals

Led flashing flash - Empty memories;

Led flashing slow - GSM module connected to cellular network;

Led flashing fast - GSM module without cell phone signal.

36.3 Programming the GSM Dialer

To program the GSM dialer follow the steps below:

8. Press "P" to enter the programming module;
9. All keys light up and the buzzer signals with 2 short beeps;
10. Enter the function "01";
11. Select the memory 1 to 6 you want to program;
12. Enter the phone number up to 16 digits;
13. Press "E" to confirm;
14. To exit the program, type "P".

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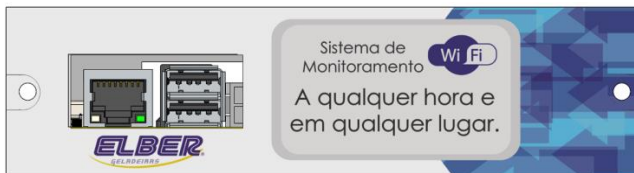
- In the case of long-distance calls, the choice of the operator and the area code is mandatory, or the call will not be completed.
- If a landline is registered, it will receive only alert alert and low battery.

To delete registered phone numbers, enter the "02" function, select the memory you want to erase (1 to 6) and press "E" to confirm.

To test the stored numbers, press the "E" key and then the memory number (1 to 6). If there is a stored number, the "E" key will remain on, starting the sending of the SMS with a bitonal alert connection. If the memory is empty, an error BIP will be issued and the test will be terminated.

For more detailed information on the operation GSM dialer, consult the manufacturer's manual at www.sulton.com.br.

37 WIFI MODULE HANDLING PROCEDURE *(see option availability)*

The WiFi module makes it possible to monitor the system via the internet. It allows you to obtain reports, system information and register mobile numbers and emails to be notified in case of problems, all through a computer with internet access.

37.1 Configuring the WiFi module

To configure the module, you will need a pen drive and a computer. Follow the steps below:

1. Insert the pen drive into your computer and create a notepad file named internet.txt. This file can not be saved inside a pen-drive folder.
2. Type in the first line the name of the WiFi network that the module should be connected to, followed by a comma and the password. Ex.:

redeelber,senhaelber

Note: Do not insert space after the comma.

3. Remove the pen-drive from the computer and insert it into one of the WiFi module's ports.
4. Wait for the display of the ELV5 controller to inform you that the network has been configured and that the pen drive can be removed.
5. The module will start sending information for remote monitoring. To obtain access to online data, send an e-mail at05@elber.ind.br, at07@elber.ind.br and at08@elber.ind.br requesting the registration.

After accessing the account, the screen below will appear, in addition to the graphs of temperature, voltage and GSM signal level, when available.

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To register numbers and emails and generate reports, follow the instructions below.

Attention!

When possible, use a network cable on the WiFi system.

NOTE: Monitoring systems that depend on the transmission and/or storage of data by Wi-fi, GPS, SMS, WhatsApp or satellite are subject to changes in technologies, licenses and possible related costs, to monitor maintenance.

37.2 History

By clicking History, you can get graph reports of temperature, battery voltage and power source. After clicking, select the desired period and click on filter. The charts and data table will appear at the bottom of the page. If desired, click Print.

37.3 Settings

The settings must be changed only by the company Elber.

37.4 Register mobile phones and emails

By clicking on e-mail it is possible to register e-mails to receive messages informing problems, like temperature incorrera and lack of electrical power. After clicking on email, click add and enter as many emails as you wish.

By clicking on mobile phones you can register mobile numbers to receive messages reporting problems. After clicking on mobile phones, click add and enter as many numbers as you wish.

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