MANUAL DE INSTRUÇÕES E GARANTIA
INSTRUCTION AND WARRANTY MANUAL
MANUAL DE INSTRUCCIONES Y GARANTÍA

CAMARA CONSERVADORA DE SEMÊN SUÍNO ELBER

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21	ELECTRICAL INSTALLATION.....	35
21.1	Static Conservatives	35
21.2	Automotive Conservatives	36
22	OPERATION.....	37
23	SOUND ALARM.....	38
24	BATTERY.....	38
25	BEST CONSERVATOR USE AND RECOMMENDATIONS	38
26	DEGREE AND CLEANING	39
27	MAINTENANCE AND PREVENTIVE VERIFICATION.....	40
28	EMERGENCY SYSTEM (see optional availability)	40
29	ELECTRONIC AUTO DIAGNOSIS (Only applicable for conservatives with an emergency system).....	41
30	PROBLEMS AND PROBABLE CAUSES	42
31	DISPOSAL OF THE PRODUCT	42
32	WARRANTY	42
33	HANDLING PROCEDURE TELE ALARM 6 NUMBERS (<i>see option availability</i>)...43	
33.1	Register phone numbers.....	43
33.2	Deleting Memory Numbers	44
33.3	Test the tele alarm memories	44
34	HANDLING PROCEDURE TELE ALARM 9 NUMBERS (<i>see option availability</i>)...45	
34.1	Registering phone numbers.....	45

34.2	Programming a Paused Phone Number 2 sec. (for PBX calls).....	45
34.3	Programming a Telephone Number for a Call Through a Economy Line	45
34.4	Programming a Long Distance Calling Phone Number	46
34.5	Deleting a Phone Number.....	46
34.6	Testing the Phone Numbers	46
34.7	Changing the Installer Password	46
34.8	Dialing Process.....	46
34.9	Dialing Interrupt Modes	47
34.10	TOTAL RESET (All dialer parameters return to Factory Default)	47
34.11	Audible Signaling (Buzzer).....	47
35	GSM HANDLING PROCEDURE (see option availability).....	47
35.1	Installing the SIM CARD.....	48
35.2	Signals.....	48
35.3	Programming the GSM Dialer.....	48
36	WIFI MODULE HANDLING PROCEDURE (<i>see option availability</i>)	49
36.1	Configuring the WiFi module.....	49
36.2	History	50
36.3	Settings.....	51
36.4	Register mobile phones and emails	51
37	CARACTERÍSTICAS DE LA CONSERVADORA	52

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DEAR USER!

You have just purchased a conservator for swine semen storage. This product has been developed with ELBER technology and functionality. We are grateful for your choice.

It is essential to read carefully the instructions contained in this manual.

Following these guidelines, you can better understand the operation of the product, taking advantage of its benefits and extending its durability. The instruction manual must always be accessible near the product so that any doubt can be remedied immediately.

19 CONSERVATOR CHARACTERISTICS

The conservative Elber has been developed to store swine semen at a stable, homogeneous and uninterrupted temperature.

Elber preservatives can operate through 110-127 VAC or 220-240 VAC or directly connected to the vehicle in 12 or 24 VDC.

The conservative ELBERs are designed to ensure that the internal chamber temperature is always stable and homogeneous. Equipped with a safety system, the conservative Elber remain operable even with a lack of commercial electric power (see optional availability). The conservative Elber has an electronic microprocessor controller that controls the internal temperature of the chamber (cooling and heating as needed) and informs the user of all events through audible and visual alarms and electronic records.

19.1 Packaging

The packaging of the product you received was designed to provide safety in the conservative transport. When unpacking, check the integrity of the product. The conservative ELBER are packed with bubble wrap and cardboard. Some products have packaging in MDF or wooden pallet. Each product is individually packaged.

After unpacking the product, all packing material should be sent for recycling or reuse.

19.2 Transportation

Conservative ELBERs must be transported in an upright position, maintaining the packaging conditions according to the label.

19.3 Storage

ELBER preservatives should be stored in clean, dry, sun-protected and excessive humidity environments.

Conservatives equipped with an internal battery should not be stored for more than two months without being used. If it is not used, the battery will discharge.

19.4 Handling

ELBER preservatives should only be handled by trained professionals who fully meet the specifications in the product instruction manual.

20 CONSERVATIVE INSTALLATION AND FIXATION

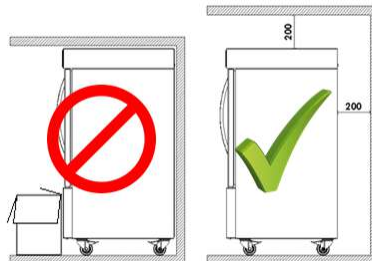
20.1 Static Conservatives

The conservator should be positioned on level and level floor. Some models have casters and other leveling feet. In conservative castors, to move the product, the casters must be unlocked and locked when the conservator is in place. Moving the guard with the castors locked may damage the castors and the floor. For conservatives with leveling feet, adjust the level after product placement.

For conservatories fitted with a water drain, connect the hose to a drain hose or a drain hose.

The conservative Elber should be installed preferably protected from the sun and in a place where there is air circulation. Very small rooms should have openings for air exchange.

We do not recommend installing the conservator near heat sources (heaters and stoves). In the conservatory ventilation racks or in the static condenser region (rear) there must be at least 20 centimeters of the wall or any obstacle to allow ventilation of the compressor unit and the electronics.

20.2 Conservative automotive (installed in cars)

The retainer should be fixed to the vehicle by means of bolts, fixed in resistant parts of the car. If necessary, the external cabinet can be used for fixing. Give preference to the corners, which have greater resistance.

ATTENTION: Do not insert drills or screws longer than 15 mm as this may affect the refrigerant piping.

For products with a specific fastening system, see instruction on the technical specifications sheet.

For conservatories fitted with a water drain, keep the drain cap closed and only open during cleaning and draining of the product.

The conservative Elber should be installed preferably protected from the sun and in a place where there is air circulation. Very small rooms should have openings for air exchange.

We do not recommend installing the conservator near heat sources (heaters and stoves). In the conservatory ventilation racks or in the static condenser region (rear) there must be at least 20 centimeters of the wall or any obstacle to allow ventilation of the compressor unit and the electronics.

It can be built into furniture, as long as it does not completely enclose the conservator and preserve the opening of the free ventilation grills for air circulation. This is necessary for there to be heat exchange with the environment and free space to remove the lid that gives access to the refrigerator compressor.

Ventilation of the compressor unit can not be obstructed. The heat exchange of the compressor unit is indispensable for the proper functioning of the product.

The ventilator flaps of the preservative are indicated on the product data sheet.

Observe the identification of air inlet and outlet. The air enters through an opening and leaves through another. This flow is promoted by the fan that is installed inside the compartment.

21 ELECTRICAL INSTALLATION

21.1 Static Conservatives

The conservator is ready to operate connected to the power of 110-127 VAC or 220-240 VAC.

Before connecting the conservator, check that the voltage indicated on the product matches the mains voltage.

The power cord must be plugged into a permanent receptacle on the wall or stand, in accordance with current regulations. Do not use extension cords or adapters.

Observe the following steps to turn on the conservator properly.

1° - The voltage 110-127 VAC or 220-240 VAC is set at the factory;

2° - Plug in the power - the conservator still will not work, because the main switch is still on;

3° - Turn on the conservator's main switch;

4° - When you turn on the conservator, the Audible Alarm will sound until the temperature inside the camera is within the ranges set in the digital controller.

You can cancel the audible alarm for a predetermined time by pressing the mute key (ELV5 and touch screen) or any key (ELV4) on the controller. When the alarm is sounding and deactivated, it will sound again after the set time if the temperature remains outside the programmed range. The alarm will sound until the conservator reaches the normal operating range (on average 2 hours without load).

To switch off the conservator, when equipped with emergency system, it is necessary to turn off the main switch. Do not simply remove the plug from the wall outlet, as the conservator will continue to operate from the battery (*see option availability*).

We recommend the use of the surge protection device (DPS) SPW 275-12 - WEG. This device protects the conservator from surges and over voltages, preventing it from burning.

Attention!

This conservator must be connected to a ground to prevent damage to the product and to the user. Ground wire connection is required at any voltage (110-124 VAC or 220-240 VAC). Lack of grounding compromises the coverage of the conservator's warranty. For grounding and electrical installation, follow the ABNT-NBR 5410 standards, or call a responsible professional.

21.2 Automotive Conservatives

The conservative Elber are designed to be powered by the vehicle's battery. Its voltage is factory set at 12, 24 or 12/24 VDC.

Check the supply voltage on the product identification label.

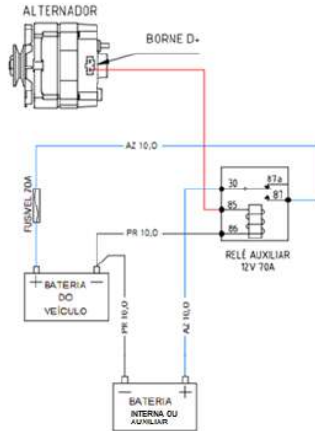
The polarity indicated on the harness must be respected. Do not consider the coloring of the wires to guide the polarity. In the refrigerator the blue color is positive and in the vehicle the blue color may be negative. Consider the (+) and (-) identification washers on the cables.

Reversing the polarity may cause the electronic unit or the protective fuse to burn.

To connect the conservative Elber, simply plug the conservator plug into the power plug. For refrigerators that do not have a plug, simply patch the cables.

For products with specific electrical connection, check the electrical connection diagram on the technical specifications sheet of the product.

For other products, it is recommended that you follow the wiring diagram below, which helps to maintain the conservator / auxiliary battery charge and does not discharge the car battery. Some conservatives have a battery charger, which can be operated with the vehicle stopped (see supply voltage on the identification label).

To install the conservator, use the electric conductor of the battery to power the refrigerator. Dimension the electric conductor (wire) according to the distance between the conservator and the battery according to the following table:

	<i>GAUGE</i>		12 V	24 V
CONDUCTOR	2,5 mm ²	For distance to	2,5 m	5 m
CONDUCTOR	4,0 mm ²	For distance to	4 m	8 m
CONDUCTOR	6,0 mm ²	For distance to	6 m	12 m
CONDUCTOR	10,0 mm ²	For distance to	10 m	20 m

The above table is indicated for products with 1 compressor, for products with more than 1 compressor, consult AT Elber to obtain the correct specification.

22 OPERATION

This conservator is prepared for the conservation of porcine semen.

It guarantees a constant temperature at any time of the year and works even when there is a lack of electricity (see optional availability). The temperature is adjusted and controlled automatically by the system and there is no need for intervention even on cold or hot days.

Ensuring uniform temperature is made by the digital controller that maintains what has been programmed. The controller is programmed to keep the chamber at **+ 15.0 ° C** to **+ 18.0 ° C** on both cold and hot days. At any time of the year, winter or summer, the conservative will adjust automatically.

The temperature will always remain in a defined range, ensuring the best properties of the preserved material.

23 SOUND ALARM

For convenience, the conservator has an audible alarm that is triggered when the temperature is outside the programmed limits or other anomalies. This can occur if the door is too long open or due to any malfunction. So in a timely manner some action may be taken so that the products in conservation are not lost. The alarm will only be activated after the preset time with a temperature outside the programmed time.

Alarm delay function: When you turn on the conservator, the alarm will sound. To suspend the alarm for the preset time, press the muting key (ELV5 and touch screen) or any key (ELV4) of the digital controller. Repeat as many times as necessary until the conservator reaches the programmed range, where it will no longer fire. To change the time between shots, see regular time function to trigger the alarm.

24 BATTERY

If the product has an emergency system, it is important to follow the recommendations below:

- For consumables with an emergency system and are purchased without the battery, always use a new battery.
- For static conservatives use stationary battery, for automotive conservatives, use automotive batteries.
- When using unsealed battery, check the level of distilled water quarterly.
- Check battery operation. If it is not functioning properly, it will cause the conservative malfunction.

25 BEST CONSERVATOR USE AND RECOMMENDATIONS

- Avoid leaving the door open.
- In use, always keep products inside the conservator, even if they are bottled with water. This load of products helps keep the temperature inside the conservator stable.
- Before putting products into the conservator, check that the operation is correct and the air circulation fan on the inner bottom is working.
- Avoid leaving the conservator door open for a long time or opening it constantly;

- It is recommended to keep the conservator running for at least 48 hours before placing products in the conservator, to check that the general operation is correct, whether the internal fan is circulating the air and if the temperature is within the set parameters;
- In use, always keep products inside the conservator, even if they are bottled with water. This load of products inside helps to maintain stable the temperature inside the conservative;
- The internal ventilation flaps can not be obstructed, constantly check if stored products are obstructing the fins and, therefore, the internal ventilation of the product;
- The shelves or drawers can not be lined with plastics, towels or the like, which compromises the internal ventilation of the product;
- On days with high humidity, air humidity (sweat) may condense on the door or on the outside edges of the conservator (near gasket). In places with room temperature controlled by air conditioning, this characteristic is almost extinct.
- For conservatives equipped with an emergency system, do not perform battery run tests without Elber technical support, face-to-face or online. Only technicians can guarantee the recording of results, disable the messaging system and ensure that the battery is not damaged (*see optional availability*).
- Only trained personnel should operate the product.

26 DEGREE AND CLEANING

To clean the conservator, first turn off at the GENERAL SWITCH. Do not simply unplug it as it will continue to run through the battery (*see option availability*).

Use damp cloths with water and mild detergent. Do not use splashes or water hoses. Excess water may compromise product performance.

Do not use chemicals (alcohol, solvents, etc.) or abrasive products and hard sponges. Non-recommended procedures may stain and damage the surfaces and coating of the refrigerator.

During cleaning, remove all materials stored in the conservator and place them in another equipment or box with cryogenic material.

Important! After a period of inactivity of the conservator (for cleaning, for example), when the power is turned on, the conservator can reach

temperatures below that programmed in the electronic controller. Therefore, whenever you turn off the conservator, after reconnecting, wait at least 2 hours to put load on the conservator again. After 2 hours, clear the controller data and start normal use.

27 MAINTENANCE AND PREVENTIVE VERIFICATION

- Monthly check that the gasket of the door gasket is correct (there must be no gaps between the gasket and the cabinet);
- Monthly check the level of glycerol present in the bottle, located inside the conservator (variable position depending on the model, some models do not have the sensor immersed in glycerol). If the sensor is out of solution, complete the level;
- For conservators equipped with an emergency system, check battery charge every six months (see optional availability);
- Conservative condenser should be cleaned annually. The location of the condenser varies for each model, but is always close to the compressor;
- Change the evaporator and condenser fans of the conservator every 4 years;
- For conservatives equipped with an emergency system, change the conservator's battery every 4 years (see optional availability).
- For conservatives installed in cars, in addition to the above checks, perform the inspections below monthly:
 - Visually check for cracks or broken parts in the conservator's structure;
 - Check the tightening of the door hinge screws by hand;
 - Visually check the integrity of the door locks. Perform practical test of the operation of the latches;
 - Visually check the integrity of the cushions and support base of the conservator (when it exists);
 - Visually verify that the electrical installation is correct (presence of rust on plugs, relays and components in general, tightening of terminals and plugs, condition of electrical cables, etc.);

28 EMERGENCY SYSTEM (see optional availability)

When equipped with an emergency system, in the absence of conventional 110-127V or 220-240V electrical power or vehicle power (12 or 24VDC), the Elber preservatives continue to operate (see autonomy in the product technical data sheet), keeping all their electrical and electronic functions in operation.

The activation of the emergency system is automatic. The autonomy of the emergency system varies for each model, can be increased or decreased, according to the need of the client.

The conservative Elber have sealed, maintenance-free and self-loading battery packs.

Battery recharge time varies between 10 and 20 hours.

Important: The autonomy of the conservator depends on many factors, such as: battery capacity used, ambient temperature, conservative load, number of times the door is opened, among others. Use as reference a 110 Ah automotive battery, which should have an autonomy of approximately 10 hours (considering ambient temperature $22 \pm 2^{\circ}\text{C}$, few door openings, new and conservative automotive battery with maximum load already at standard temperature). The higher the battery amperage, the greater the autonomy of the conservator.

The emergency system is optional, check the availability of the optional.

Some models can be sold with the emergency system, however, without the battery.

29 ELECTRONIC AUTO DIAGNOSIS (Only applicable for conservatives with an emergency system)

LED	Blinking	Diagnosis
Blinking	1X	Battery cut-off (voltage not specified)
Blinking	2X	Many attempts to start the compressor or short fan (current above 0.5A)
Blinking	3X	Compressor did not break or over pressure. Wait 4 minutes for it to automatically reconnect.
Blinking	4X	Minimum compressor rotation error
Blinking	5X	Ambient temperature too high
Blinking	6X	Thermostat is shorted or the electronic unit is incorrectly connected.

30 PROBLEMS AND PROBABLE CAUSES

FAILURE	SOLUTION
Conservative does not turn on	<ul style="list-style-type: none"> - Check that the supply voltage is the same as that of the conservator; - Check for power in the power supply; - Check that the main switch is switched on; - Check that the power plug is securely connected; - Check that the fuse is blown; - Check that the set temperature in the controller is correct; - Check if there is any warning on the digital controller.
Conservative does not cool;	<ul style="list-style-type: none"> - Check that the compressor is switched on (place your hand on the compressor and check for vibration and slight heating); - Check that the internal ventilation is working; - Check the door closing (door gasket must not have cracks in relation to the cabinet); - Check if there is any warning on the digital controller.
Conservative alarm by high temperature	<ul style="list-style-type: none"> - Check that a large load of product has been charged within 2 hours (the conservator can take up to the ideal temperature when a large quantity of product is placed at one time); - Check door closing (door gasket must not have cracks in relation to cabinet). - Check if there is any warning on the digital controller.
Conservative internal temperature varying.	<ul style="list-style-type: none"> - Check that the internal fans are working; - Check that the internal ventilation flaps are free (they can not be obstructed);

31 DISPOSAL OF THE PRODUCT

Disused or abandoned refrigerators can be dangerous, especially for children, as they can get stuck inside, running the risk of shortness of breath. Before discarding your old refrigerator, cut off the power cord and remove the door (s). Leave the shelves in place so the children can not enter.

If the conservator's battery is discarded, route the discarded battery to a nearby pickup point.

In case of product disposal, all materials should be sent for recycling.

32 WARRANTY

Conservative warranty is 1 year.

Attention: The warranty term for the battery is included with it, and can not be misplaced as it contains the serial number that matches the battery number (see option availability).

33 HANDLING PROCEDURE TELE ALARM 6 NUMBERS (see option availability)

The tele alarm operates through a telephone line. It makes calls to the registered numbers when the temperature exits programmed range, or if the battery voltage is low. For the dialer to work, you need to connect a telephone cable.

33.1 Register phone numbers

The tele alarm has memory to register up to 6 phone numbers, with up to 16 digits each.

To register a phone number, press the "PROG" key to enter programming mode. Enter "1", which corresponds to the desired function, and then press the number of the memory the phone number should occupy, from 1 to 6. Enter the phone number and press "ENTER" to confirm. Press "ENTER" again to exit programming mode. Ex.:

When the line has a PABX extension, press "0" to pick up the external line, and the "PROG" key to enter a 2-second pause before the telephone number. Ex.:

If a number is stored in an already occupied memory, the new number will replace the previous one, which will be deleted automatically.

We recommend that you record numbers from at least two different carriers, if the service provider fails and you can not perform one operation, the dialer can complete the call to another.

In case of tele alarm triggering, it will call all the numbers, according to the registration sequence. The dialer will continue to make the call sequence while there are shots.

33.2 Deleting Memory Numbers

Press the "PROG" key to enter programming mode. Enter "2", which corresponds to the desired function, and then press the number of the memory that corresponds to the phone number you want to delete (from 1 to 6).

Press "ENTER" to confirm, and again to exit programming mode. Ex.:

33.3 Test the tele alarm memories

Press the "ENTER" key and then the memory number corresponding to the phone number you wish to test. The LED will blink twice, ascend, and initiate the test call. If the memory is empty, the LED will rise for 1 second and the test will be completed. Ex.:

To cancel the test press the "PROG" key.

THE OTHER FUNCTIONS OF THE TELE ALARM ALREADY COME PROGRAMED FROM THE FACTORY, NOT NECESSARY TO CHANGE.

34 HANDLING PROCEDURE TELE ALARM 9 NUMBERS *(see option availability)*

The alarm phone works over a telephone line. Calls the registered numbers when the temperature is outside the programmed range, or if the battery voltage is low. For the dialer to work, you need to plug in a phone cord.

34.1 Registering phone numbers

Press # + Installer Password + Index (01 to 09) + Phone Number (up to 22 digits) + # + #.

34.2 Programming a Paused Phone Number 2 sec. (for PBX calls)

Press # + Installer Password + Index (01 to 09) + 0 + E + Phone Number + # + #.

If you need to use the characters * (asterisk) or # (pound), proceed in the same way by pressing respectively the P or F key in the position where they are to be entered.

34.3 Programming a Telephone Number for a Call Through a Economy Line

For calls of this type, where you need to enter * 15 seconds for a 6-second pause before dialing, press # + Installer Password + Index (01 to 09) + P + 15 + E + E + E + Phone Number + # + #.

Phone numbers longer than 22 digits are not allowed. When the 22nd digit is entered, the system automatically records its value and does not need to press # to store it.

34.4 Programming a Long Distance Calling Phone Number

Press # + Installer Password + Index (01 to 09) + 0 + Operator Number + Phone Number + # + #.

34.5 Deleting a Phone Number

Press # + Installer Password + Number Position you want to delete (Index 01 to 09) + * + # + #.

Another way to delete a preset number is to access the desired index and omit the phone number value by pressing #. Be careful when accidentally accessing phone numbers (01 to 09) as you will need to re-enter their value.

34.6 Testing the Phone Numbers

With the telephone line correctly connected and the dialer in user mode (out of programming), press F + desired Memory Location (1 to 9 - without digit 0).

The dialer will make a call to the stored number, playing a siren sound on the phone for 60 seconds. During this time, the LINE LED will remain lit. To stop an ongoing test, press any key.

34.7 Changing the Installer Password

The installer password is for accessing Programming Mode and its factory default is 4321. To change, press # + Installer Password (current) + 10 (index) + New Installer Password (4 digits) + #.

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34.9 Dialing Interrupt Modes

Concurrent Mode: The dialer will make calls only while the alarm or fence goes off.

Persistent Mode: The dialer will make calls even if the trigger no longer occurs, completing the 2 cycles.

Press # + Installer Password + 12 (index) + 1 (Concurrent Mode) or 2 (Persistent Mode) + #.

34.10 TOTAL RESET (All dialer parameters return to Factory Default)

Turn off the dialer power and hold down the 7 and 9 keys. Temporarily reset the dialer until you hear 3 short beeps. Release keys 7 and 9 and power up the dialer.

34.11 Audible Signaling (Buzzer)

1 Long beep = Error tone (incorrect command or procedure).

1 Short Beep = Indicates a key press.

3 Short Beeps = Confirmation Tone (command or procedure accepted).

THE OTHER FUNCTIONS OF THE TELE ALARM ALREADY COME PROGRAMED FROM THE FACTORY, NOT NECESSARY TO CHANGE.

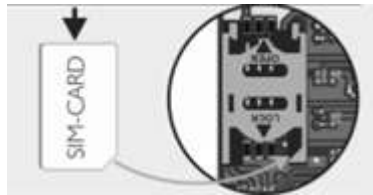
35 GSM HANDLING PROCEDURE (see option availability).

The conservative Elber can be equipped with GSM dialer, which sends SMS messages and makes calls in case of events. Below are the main instructions related to this item.

35.1 Installing the SIM CARD

Remove the front cover of the GSM dialer by pressing a small latch on the bottom of the carton. When removing the front, do not disconnect the cables. Choose a SIM CARD from an operator of your choice and enter as indicated below:

- Only the MICRO SIM CARD model can be used.
- Never remove or place the SIM CARD with the GSM dialer on.
- The SIM card does not come with the dialer. Choose an operator that you trust because the operation of the device is directly related to the cellular telephone network in your region.

35.2 Signals

Led flashing flash - Empty memories;

Led flashing slow - GSM module connected to cellular network;

Led flashing fast - GSM module without cell phone signal.

35.3 Programming the GSM Dialer

To program the GSM dialer follow the steps below:

8. Press "P" to enter the programming module;
9. All keys light up and the buzzer signals with 2 short beeps;
10. Enter the function "01";
11. Select the memory 1 to 6 you want to program;
12. Enter the phone number up to 16 digits;
13. Press "E" to confirm;
14. To exit the program, type "P".

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- In the case of long-distance calls, the choice of the operator and the area code is mandatory, or the call will not be completed.
- If a landline is registered, it will receive only alert alert and low battery.

To delete registered phone numbers, enter the "02" function, select the memory you want to erase (1 to 6) and press "E" to confirm.

To test the stored numbers, press the "E" key and then the memory number (1 to 6). If there is a stored number, the "E" key will remain on, starting the sending of the SMS with a bitonal alert connection. If the memory is empty, an error BIP will be issued and the test will be terminated.

For more detailed information on the operation GSM dialer, consult the manufacturer's manual at www.sulton.com.br.

36 WIFI MODULE HANDLING PROCEDURE *(see option availability)*

The WiFi module makes it possible to monitor the system via the internet. It allows you to obtain reports, system information and register mobile numbers and emails to be notified in case of problems, all through a computer with internet access.

36.1 Configuring the WiFi module

To configure the module, you will need a pen drive and a computer. Follow the steps below:

6. Insert the pen drive into your computer and create a notepad file named internet.txt. This file can not be saved inside a pen-drive folder.

7. Type in the first line the name of the WiFi network that the module should be connected to, followed by a comma and the password. Ex.: *redeelber,senhaelber*

Note: Do not insert space after the comma.

8. Remove the pen-drive from the computer and insert it into one of the WiFi module's ports.
9. Wait for the display of the ELV5 controller to inform you that the network has been configured and that the pen drive can be removed.
10. The module will start sending information for remote monitoring. To obtain access to online data, send an e-mail to at04@elber.ind.br and at05@elber.ind.br requesting the registration.

After accessing the account, the screen below will appear, in addition to the graphs of temperature, voltage and GSM signal level, when available.

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To register numbers and emails and generate reports, follow the instructions below.

Attention!

When possible, use a network cable on the WiFi system.

36.2 History

By clicking History, you can get graph reports of temperature, battery voltage and power source. After clicking, select the desired period and click on filter. The charts and data table will appear at the bottom of the page. If desired, click Print.

36.3 Settings

The settings must be changed only by the company Elber.

36.4 Register mobile phones and emails

By clicking on e-mail it is possible to register e-mails to receive messages informing problems, like temperature incorrera and lack of electrical power. After clicking on email, click add and enter as many emails as you wish.

By clicking on mobile phones you can register mobile numbers to receive messages reporting problems. After clicking on mobile phones, click add and enter as many numbers as you wish.

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